

# TURNOUT™ with CARONY™

A BRUNO TAS™ PRODUCT

The transport wheelchair that increases your freedom!

**Comfortable transportation without lifting! Simple to maneuver, push and load.**



1

The Carony is easy to push and maneuver, especially in small areas.



2

The Carony Seat transfers easily from the base to the Turnout. Start by rotating the **TURNOUT**.



3

Remove the push handle and footrest and connect the seat to the **TURNOUT**. **LOCK THE BRAKE.**



4

Release a latch on the seat and push the seat into the car. Disconnect the wheelchair base from the **TURNOUT**.



5

Rotate the seat into position, be sure that the seat locks and fasten the seat belt. The transfer is now complete.



6

Store the transport wheelchair base in the trunk and make sure it is secure.

## HEADREST

Comfortable and removable.

## SEATING

The Bruno SM Seat provides lumbar support as well as lateral support in both the seat cushion and backrest. The seat and back cushion inserts are removable for cleaning.

## FOOTRESTS

Adjustable, removable and swing-away.

## ADJUSTABLE HEIGHT FRAME

Crank handle allows adjustment over a range of 4.5" (115mm) to adapt to the vehicle, ground condition and user weight.

## AMPLE FOOT CLEARANCE

The U-shaped frame provides ample foot clearance during the transfer.

